TCA 6
MOBILE IMAGING SYSTEM

Versione "Compact" con monitor installato a bordo macchina, senza carrello portamonitor.
"Compact" version with monitor mounted on the "C", without monitor trolley.

Tastiera ergonomica con rotazione +/-60° composta da display graico touch screen.
Ergonomic and +/-60° rotating keyboard with touch screen graphic display.

Ruote orientabili (0°-45°-90°) + freno
Orientable wheels (0°-45°-90°) + brake.

Azionamento extra corsa verticale.
Extra vertical run push button.

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The TCA6 is a highly optimised mobile System for surgical luoroscopy. The optimisation is the pursuit of the best compromise within the peculiar features of such an instrument: dimensions, weight, balancing, ease of manoeuvre, X-ray features, image quality, safety.

The large achievable distance of the SID axis from the unit's stand, the noticeable motorised column elevation, reaching 500 millimetres, the wide C-arm's insertion and span spaces, the ergonomic $\pm 60^\circ$ rotating keyboard, enable the best operational flexibility and satisfy the need of the user in the operating theatre.

Sala operatoria della Clinica Humanitas Gavazzeni. Clinica Humanitas Gavazzeni's operating theatre.

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Moreover the "Compact" Version, without the monitor trolley, but with the LCD monitor mounted on the "C", allows further reduction of the volumes in the cones of the sterile area.

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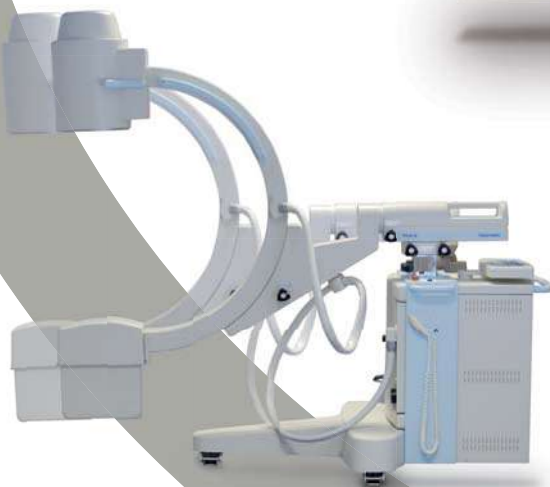
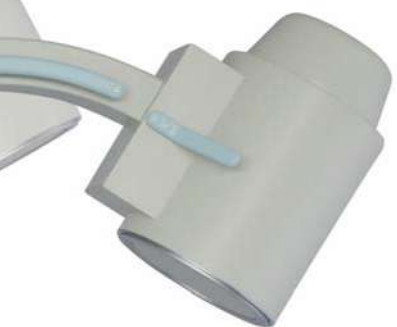
The TCA6 can be equipped with X-Ray monobloc with fixed or rotating anode and with 9" or 12" I.I. tube.

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The TCA6 can be fitted with a LASER targeting system that, allows the identification of the beam centre, through a light cross: by means of this device, the preliminary positioning of the fluoroscopy system is possible, without x-ray emission.

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The TCA6 can be equipped with 1Kx1K CCD Camera and image memory banks, of 1 (LIH), 330, 2700, , non volatile frames. All the memories are equipped with their own keyboard for data patient editing.

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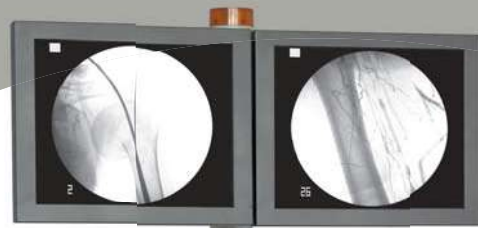
Furthermore the unit can be fitted with the Digital Imaging System for the processing of the fluoroscopic image: the large capacity of its main memory element, (of more than 40,000 frames), enables the storage of the produced diagnostic/operative documentation in the very many working conditions.

Pedale per luoroscopia.
Fluoroscopy footswitch.

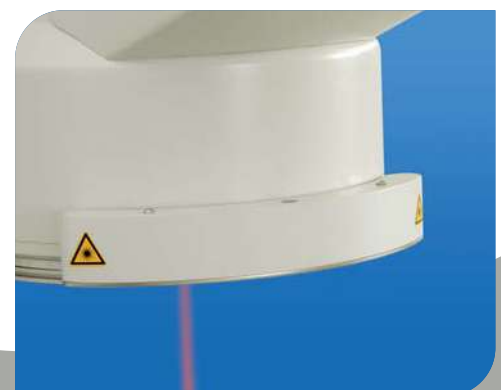
Stampante Dosimetro.
DAP (Dose Area Product) Printer.

Spingicavo.
Cable pusher.

Centratore Laser.
Laser Targeting Device.

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It exists, in addition, the possibility to input text descriptions in the image sequences, about the all data relevant to the hospital, physician, department, patient and to the examination itself. It is also possible, as an optional, to send, into the hospital DICOM standard data network, the obtained and recorded images/sequences, or to store them on a permanent archive, as CD-ROM. An optional interface of type DICOM-Printer, enables the forwarding of the images, to any printing station connected to the DICOM network.

■ ■ ■ *The monitor trolley is easily separable from the main C-arm stand by means of a connector. The monitor trolley, once disconnected from the main stand, can be taken outside of the operating theatre, enabling the reviewing of the all recorded sequences and the performing of further post-processing procedures.*

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■ ■ ■ *A scrupulous unit's design gives the possibility, in the everyday use, of an easy, simple and efficient sterilisation of the equipment's surfaces and of a fast and safe placement/removal of the sterile covers.*

TECHNIX S.p.A.
24050 Grassobbio
Bergamo - Italy
Via E. Fermi, 45
Tel. +39.035.38.46.611
Fax +39.035.33.56.75
technixd@technix.it
<http://www.technix.it>